


# Using the FIT (Fecal Immunological Test)

Collection Instructions:  
No preparation is required and  
there are no dietary restrictions.

This FIT package  
includes:

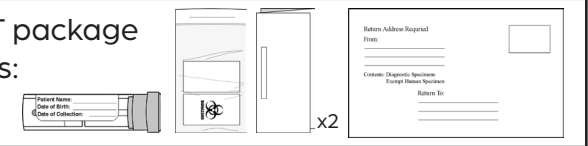

**1**

Patient Name:  
Date of Birth:  
Date of Collection:

**! IMPORTANT!**  
Write your **name, date of birth, and date of collection** on the sample collection tube.

**2**

Lay sample collection paper across toilet bowl.  
Secure adhesive tabs and then lower toilet seat.

**3**

Deposit stool on collection paper.

**4**

Open the sample collection tube.

**5**

Poke spiral applicator into stool at **6 (SIX)** different locations.

**6**

Close the sample collection tube.

**7**

Insert sample collection tube into specimen pouch.  
Then insert specimen pouch into return mailer and seal.

Return Address Required  
From: \_\_\_\_\_  
John Doe  
123 Main Street  
Anytown, USA 12345  
Contents: Diagnostic Specimens  
Return To: \_\_\_\_\_  
Your Laboratory  
123 Main Street  
Anytown, USA 12345

**8**

Return immediately via mail or in person at your  
sample drop off location.

### Results may be invalid if:

- Too much stool is collected
- Sample is frozen or overheated
- Feces touches water or urine in the toilet bowl
- The sample is returned after 8 days

### For assistance with these instructions, please contact:

- Your physician's office  
OR
- Hemosure Technical Support:  
1-888-HEMOSURE (436-6787)  
E-mail: [techsupport@hemasure.com](mailto:techsupport@hemasure.com)