



TÈS HEMOSURE® iFOB

-Tès Feko Okult Imunochimik (Immunochemical Fecal Occult Test, FIT)



Chwa ki klinikman ak finansyèman kòrèk pou Tès San Feko Okult

“ACG la sipòte rekòmandasyon gid mityèl la pou abandone ansyen tès san feko okult ki baze sou guaiac kòm yon metòd pou depistaj CRC ... ACG la rekòmande FIT kòm tès deteksyon kansè prefere a.”

Reenprime avèk pèmisyon Macmillan Ltd Publishers: Jounal Ameriken an sou Gastroenteroloji (The American Journal of Gastroenterology) (2320970223955), dwadotè (2009)

Rezime & Konparezon

| | HEMOSURE® FDA CLEARED & CLIA WAIVED | TÈS TRADISYONÈL POU GUAJAC |
|--|--|---|
| Metòd Tès | Tès imunolojik | Tès chimik |
| Entèpretasyon | Rezilta klè ak egzat ki konparab avèk tès gwosès | Koulè ble, difisil souvan pou entèprete |
| Spesifisite Gastwoentestinal ki pi ba | Plis pase 96%. Espesifik anpil pou emogloblin imen | Pòv. Rezilta pozitif ki fo akòz manje ak medikaman |
| Sansiblite Anjeneral | Plis pase 87%. Detekte osi ba ke 50ng hHB / mL imèn emogloblin espesifik | 50%. Detekte 90,000 ng hHB / mL oswa pi wo nan emogloblin ki pa espesifik |
| Presizyon | Plis pase 97% | Mwens pase 86% |
| Restriksyon oswa Preparasyon Dyetetik oswa nan Dwòg | OKENN | WI. Plizyè jou |
| Echantiyon ki Nesesè | YOUN ki obligatwa | TWA ki obligatwa |



TÈS HEMOSURE® iFOB

-Tès Feko Okult Imunochimik (Immunochemical Fecal Occult Test, FIT)




Remak: Pa gen Restriksyon Dyetetik oswa Dwòg

Enstriksyon pou Pasyon

ETAP 1

Depo Echantiyon


A Leve chèz twalèt la epi mete echantiyon papyè koleksyon an sou bòdi ekstèn twalèt la. Ranje onglè adezif sou toude bò bòdi ekstèn twalèt la. Bese chèz twalèt la.

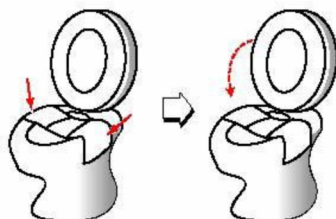


ETAP 2

Koleksyon Echantiyon


A Devise bouchon vyolèt la nan echantiyon tib koleksyon an. **PA JETE LIKID LA.**

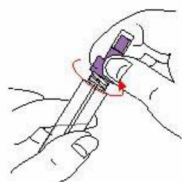

B Pike aplikatè espiral la nan pou pou a nan 6 kote diferan. Sevi sèlman avèk ase materyèl fekal pou kouvri pwen aplikatè a. **PA FÔME TOUF, RANMASE OSWA PLEN TIB LA.**

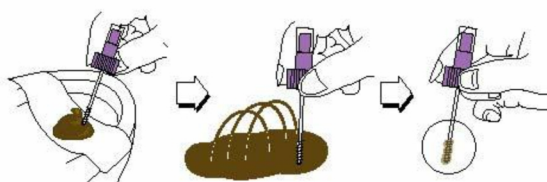

C Vise aplikatè a nan tib la epi sere li byen fo.

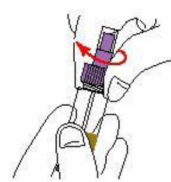

ETAP 3

Remèt Echantiyon

A Ranpli adrès la epi retounen li bay doktè w oswa laboratwa w.



B Mete tib koleksyon echantiyon an nan sak espesimèn la epi fèmen li. Mete sak espesimèn la nan anvlòp la ki te founi epi fèmen li.



C Retounen pakè echantiyon an imedyatman bay klinik la oswa laboratwa a swa pa lapòs oswa anpèsonn.



HEMOSURE, INC



Anblèm Hemosure se yon mak anrejistre nan Hemosure, Inc.